| CareHealth Meditech

Sizing Guide

Built by Dentists, for Dentists.

Available Configurations

AllIR Portable is always delivered and shipped in Configuration A. Configuration A is suitable for operatories that are average sized (10' X10') or larger. Two additional accessories are available for purchase that deliver additional configurations (Configuration B and Configuration C). The purpose of these additional configurations is to provide options for clinicians that practice in a smaller than average sized. Choice of configuration does not impact performance of the device.

Configuration A “Standard”




AllIR is purchased and is shipped in configuration A. This configuration allows the device to be placed at the foot of the chair. It is suitable for average sized or larger than average sized operatories.

Configuration B

AllIR is purchased and shipped in configuration A, and **90 degree collection dome is purchased** and shipped separately. This configuration allows stage 3 of the modular arm to be operated in a vertical position. This configuration is suitable for operatories that are slightly smaller than average, or in large operatories where cabinet access or other reasoning drives placement of device more towards chairside as opposed to the foot of the chair.

Configuration C

AllIR is purchased and shipped in configuration A, and **vertical stage 1 is purchased** and shipped separately. This configuration allows the device to be placed chairside and close to the patient’s head. This configuration is suitable for operatories that are built much smaller than average. Although device performance is not impacted, patient experience is deteriorated slightly as device is now closer to the patient and although not actually more loud, the device will seem louder to the patient due to its proximity.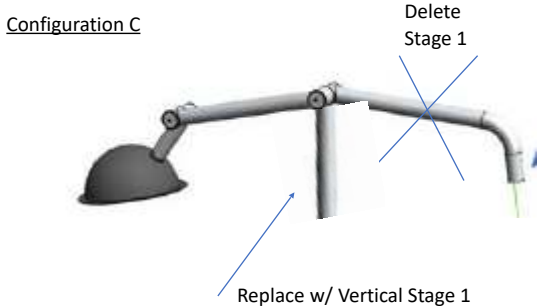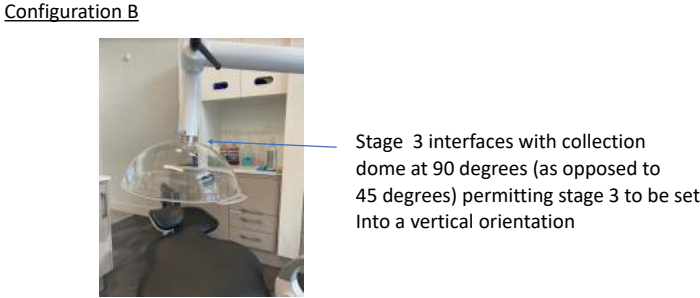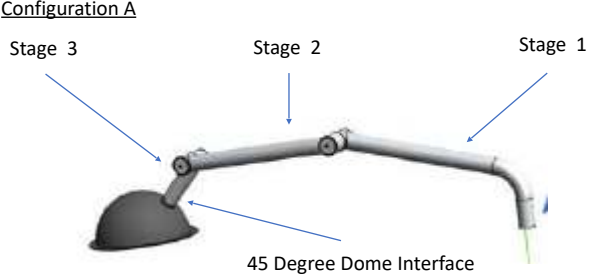

Sizing and Pre-Installation Instructions

Step 1. From a piece of cardboard or poster-board, cut out a square that is 22" X 22". This square represents the footprint of the AIIR portable cabinet. Draw straight lines "corner to corner" of the template, forming an "X". The center of the "X" is the center of the cabinet.

Step 2. Using this template as a visual cue, locate a desirable location in your operatory for the AIIR base cabinet. This location must be within AIIR's operational range of 36 to 72 inches as measured from the center of the cabinet to the patient's mouth. The best location is one that places the AIIR cabinet at the foot of the chair with as little interference with millwork and other equipment in the operatory as possible. Please note that AIIR base cabinet is 38 inches tall. Use a standard household tape measure to check vertical clearances. If you are unable to find a suitable location for the base cabinet, your operatory may require reconfiguring, or it may be too small for AIIR.

Step 3. Using a standard tape measure, measure the distance from the center of the cabinet template to the position of where your patient's mouth would be at the head of the chair. Note this distance. If the distance is less than 36" you will need to find a different location for the base cabinet and repeat step 2 and 3 as this chosen location for the AIIR cabinet is inside of the operational range of the modular arm.

Step 4. Using the distance calculated in step 3, define which AIIR configuration is best for your space (see next page for configuration advice).


NOTE: Average or larger size operatory will have several locations that are suitable for the placement of AIIR. Challenge presents when smaller than average sized operatory are chosen to install AIIR.

For smaller operatory it is critical that pre-installation measurements be taken and templates used to avoid AIIR purchase when it will not fit. As use of this pre-installation sizing guide will eliminate purchase of AIIR for operatory that are too small, Care Health Meditech will not accept returns for devices purchased for operatory that are too small.


Configuration Choice

After determining the desired location for the cabinet and using the measurement derived in step 3 of sizing and pre-installation instructions, determine the AIIR configuration that works best for your space. Note that in operatories that are of average size or greater (+/- 75% of operatory) the standard configuration A is generally suitable. Alternate configurations are required only for installation in operatory that are smaller than average or full of other equipment.


Scenario 1: If the distance from the center of the cabinet to the patient's mouth is **67" or greater**, the standard configuration A is suitable for your operatory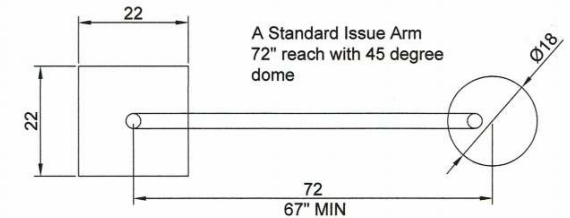

Scenario 2: If the distance from the center of the cabinet to the patient's mouth is between **59" and 66"**, configuration B (90 degree dome) is suitable for your operatory

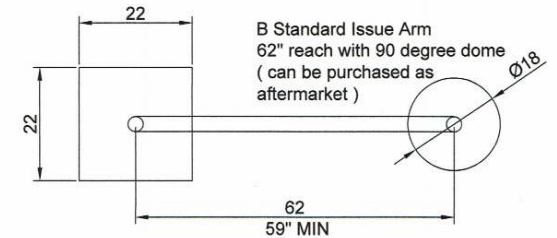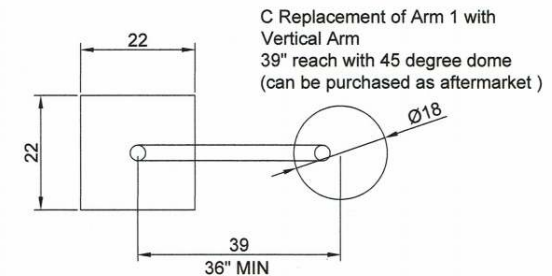Scenario 3: If the distance from the center of the cabinet to the patient's mouth is between **36" and 58"**, configuration C (vertical stage 1) is suitable for your operatory. If you cannot find a location for the cabinet that is FURTHER than 36" from the center of the cabinet location to the patient's mouth, your operatory may be too small, and AIIR purchase should not be made without further consultation

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